



Cartridge Replacement

# STEP 1

## Removing Cartridge.



Turn off the X-PLO<sub>2</sub>R™ by pressing the power button for 2 seconds.



Remove the X-PLO<sub>2</sub>R from the carrying bag, if applicable.



Remove the battery from the X-PLO<sub>2</sub>R.



Place the X-PLO<sub>2</sub>R on its side to access the underside of the POC.



Pull the cartridge out of the POC by pulling on the cartridge pull handle.




Remove the cartridge completely from the X-PLO<sub>2</sub>R and throw away.



Cartridge Replacement

# STEP 2

## Installing Cartridge.

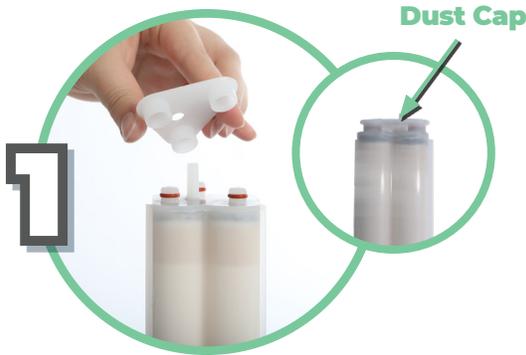

1 Remove cartridge from mylar bag. The new cartridge has a dust cap on top. Remove the dust cap. Make sure there is no dust or debris where the dust cap was located.



2 Insert new cartridge top first into the X-PLO<sub>2</sub>R. Do not leave the cartridge ends exposed; the cartridge should be inserted into the X-PLO<sub>2</sub>R as soon as the used cartridge is removed.



3 Push the cartridge into the X-PLO<sub>2</sub>R so it is fully inside.



4 The cartridge pull handle should lay flat once the cartridge is secured.



5 Connect charged battery to the X-PLO<sub>2</sub>R.



6 Turn on the X-PLO<sub>2</sub>R by pressing the power button for 2 seconds. Set the flow setting to your prescribed setting and use normally.

