


OxyHome® Quick Reference Guide

1. Indicator Lights

There are 4 indicator lights on the device:

1. Low outlet pressure indicator (Low Pressure): Outlet pressure is less than 2.2 psi, the indicator light will flash yellow every two seconds and an alarm will sound.
2. Low oxygen purity indicator (Low Conc.): If the oxygen concentration drops below 82% (V/V) the indicator light will flash yellow, and the alarm will sound.
3. Power loss indicator (Power Outage): When there is a power outage, the indicator light will flash yellow every 2 seconds and the alarm will sound.
4. Power status indicator: When the concentrator is powered on, the indicator light will remain solid green if the concentrator is working normally.

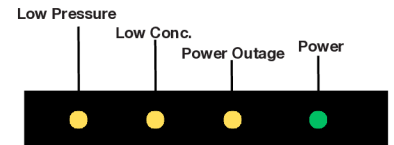

2. Alarms

Item No.	Alarm type	Priority	Alarm Signals / Indicator light			Auditory Yes/No	Alarm condition delay/s	Alarm signal generation delay/s
			Color	Flashing frequency	Vacancy ratio			
1	Power supply failure alarm	Med	Yellow	Flashing every two seconds	50%	Yes	5s	5s
2	Start-up fail alarm	Med	Yellow	Flashing every two seconds	50%	Yes	5s	5s
3	Low oxygen concentration ALARM (non-startup)	Med	Yellow	Flashing every two seconds	50%	Yes	5 s	5s
4	Pressure failure alarm	Med	Yellow	Flashing every two seconds	50%	Yes	5 s	5s
5	No flow alarm	Med	On screen	Flashing every two seconds	N/A	Yes	5 s	5s
6	Low flow alarm	Med	On screen	Flashing every two seconds	N/A	Yes	10 s	5s

3. Replacement Filters

Part Number	Description
1170-5401	Replacement foam cabinet filter for OxyHome 5 Liter Stationary Concentrator
1170-5402	Replacement HEPA/Compressor inlet filter for OxyHome 5 Liter Stationary Conc.



#1400-5100